TST Single Use Bowie Dick Test Pack - Instructions For Use

A steam penetration test for daily use as required in EN ISO 17665-1

For use in Euronda autoclaves of new generation
STEAM Sterilizer using cycle at 134°C for 3.5 minutes

Complies to: EN867-5 Type 2

Instructions for Use

1. Pre-heat the sterilizer.
2. Record relevant test cycle information on the outside of the pack in the space provided (if required).
3. Place the TST Pack horizontally on the lower shelf of the preheated empty sterilizer.
4. Run a cycle of the machine, using the BD/Helix Test Cycle from the service menu at 134°C for 3.5 minutes. The TST Pack should be removed immediately from the sterilizer and the results checked. Care should be taken, as the TST pack may be warm.
5. If the sterilizer air removal stage is functioning correctly and enabling rapid and even steam penetration, the ink pattern on the TST Pack indicator sheet will have changed from yellow to dark blue / purple colour.
6. If even and rapid steam penetration did not occur, the ink pattern on the TST Pack indicator sheet will show a yellow or brown colour towards the centre of the pack. Refer to Colour Guide right.
7. The information should be recorded on the TST Pack indicator sheet, which should then be filed in a cool, dry, dark place away from direct light.

Interfering substances or conditions known to affect the performance of the indicator

After a period of 24 – 48 hours, the blue/purple of the test sheet may darken. Refer to Colour Guide right. This is completely normal and is due to the test sheet drying out after processing.

Also after a similar period, the blue/purple of the indicator sheet may assume a grey colour. This is due to over-processing i.e. long heating up times and/or temperature / times exceeding the stated working conditions of the pack, which may invalidate the test.

If the processed indicator sheets are exposed to strong light, some fading of colour may occur.

Note: Serious incidents that have occurred in relation to this medical device should be reported to the manufacturer and competent authority in the country where the incident occurred.

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Albert Browne Ltd
STERIS Ireland Ltd